

Item information

Colouring paper





Colouring paper Unicorn
on velcro and fleece tape



Colouring paper Unicorn
on Streifylene



Colouring paper Zebra
on Streifylene



Colouring paper Robot
on Streifydur Ortho



Colouring paper Zebra
on polyester belt



Colouring paper Travel
on Streifydur

Outstanding quality, surprisingly easy to process

Effortless

The colouring paper is easy to apply, even with folds or creases. Which means: **folded storage?** No problem!

No matter which design, the **processing remains the same.**
Easy to peel off – even without silicone spray.

Vibrant

An intensive colour transfer creates an **optimal result.** Thanks to the special manufacturer's process, subsequent staining is reduced enormously – brilliant colours for **satisfied customers.**

Sustainable

For climate protection and the environment: our colouring paper now uses **even less ink and paper** in production.


Versatile

Bring colour to white **thermoplastics** and **textiles**, for example:

- Streifylene
- Streifydur
- Streifydur Ortho
- Hook and loop strap
- Polyester belt
- Laminating resin materials and many more

What is your favourite design?


All colouring papers have a dimension of 1,000 x 1,600 mm
The paper weight is 20 g/m².

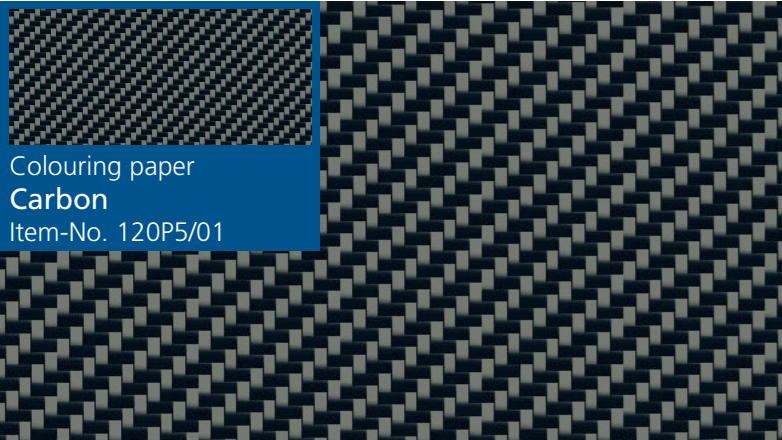

Colouring paper
Carbon
Item-No. 120P5/01




Colouring paper
Butterflies
Item-No. 120P5/02

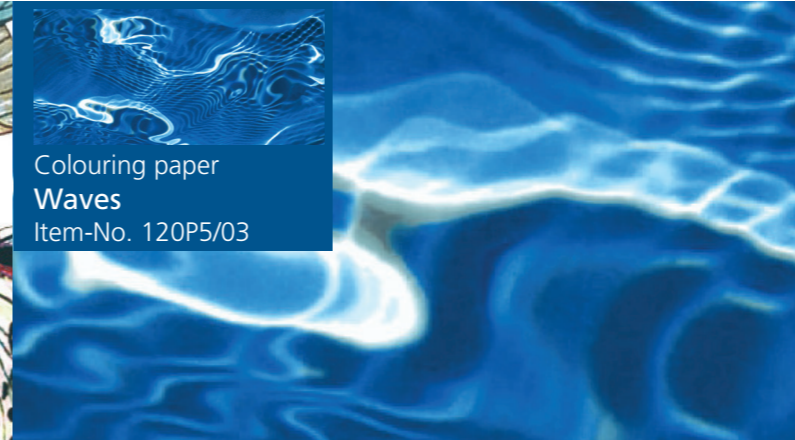

Colouring paper
Waves
Item-No. 120P5/03



Colouring paper
Palm trees
Item-No. 120P5/04

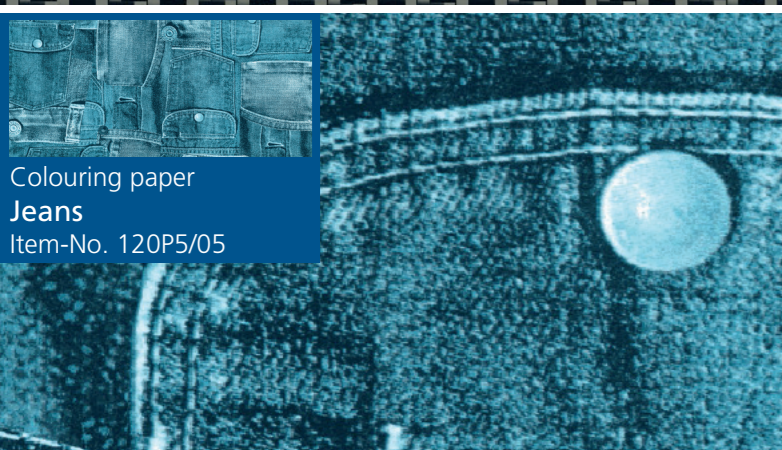


Colouring paper
Jeans
Item-No. 120P5/05




Colouring paper
Pink
Item-No. 120P5/06




Colouring paper
Camouflage
Item-No. 120P5/07

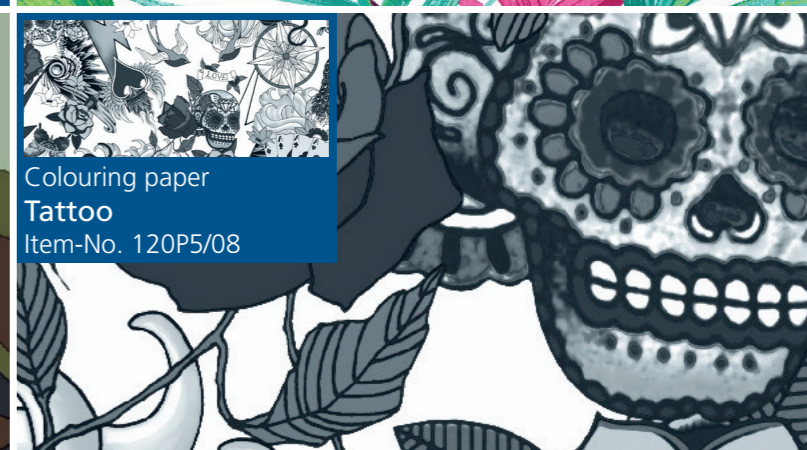


Colouring paper
Tattoo
Item-No. 120P5/08

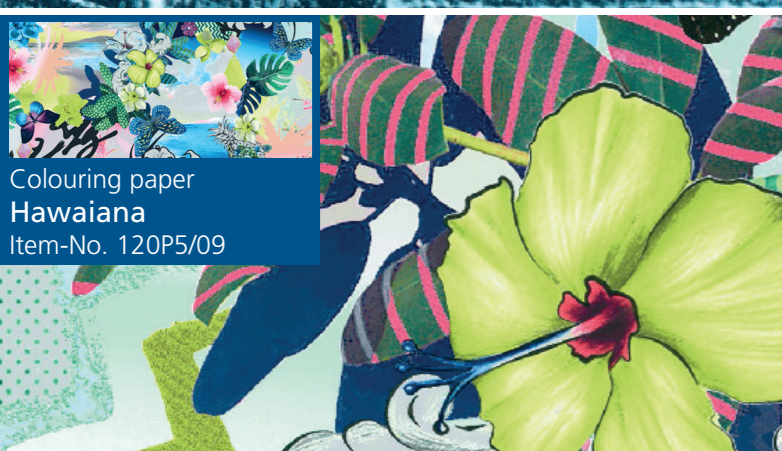

Colouring paper
Hawaiana
Item-No. 120P5/09

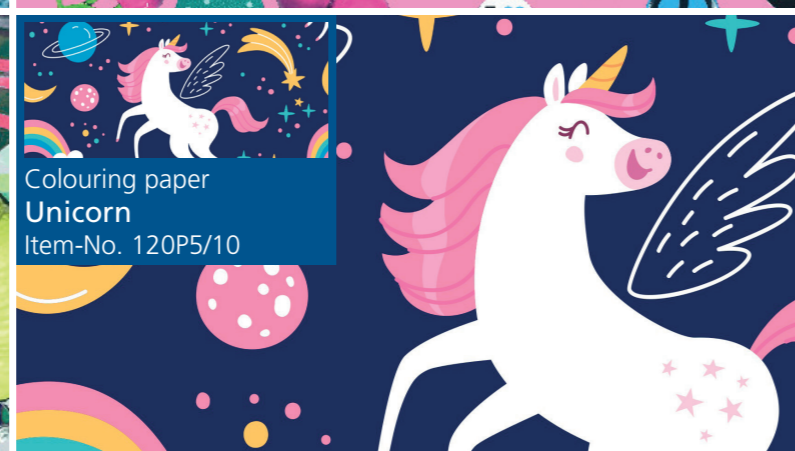

Colouring paper
Unicorn
Item-No. 120P5/10

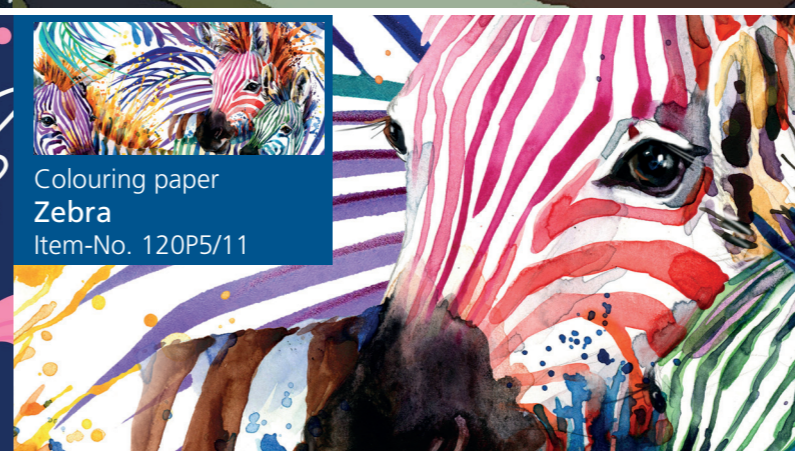

Colouring paper
Zebra
Item-No. 120P5/11




Colouring paper
Dinosaur
Item-No. 120P5/12



Colouring paper
Nature
Item-No. 120P5/13



Colouring paper
Travel
Item-No. 120P5/14

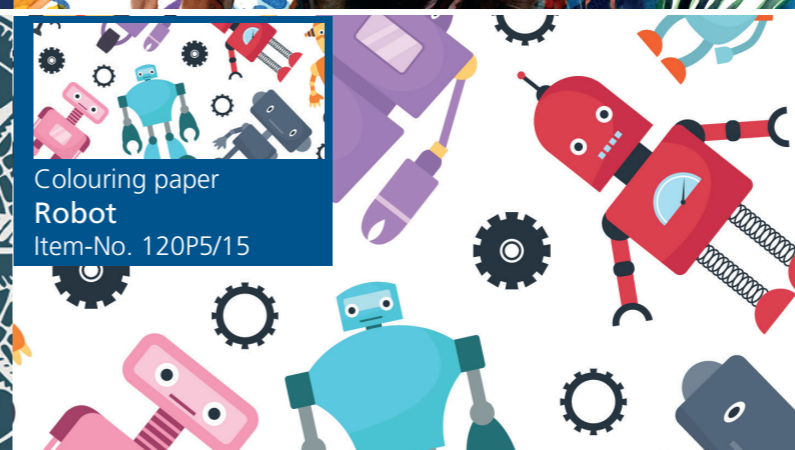

Colouring paper
Robot
Item-No. 120P5/15

- Colouring paper **Black** Item-No. 120P5/16
- Colouring paper **Skin** Item-No. 120P5/18
- Colouring paper **Brown** Item-No. 120P5/20

- Colouring paper **Blue** Item-No. 120P5/17
- Colouring paper **Red** Item-No. 120P5/19



Tips for an optimal result with colouring paper



Processing tips

Heat the cut thermoplastic completely to its specified processing temperature (min 140°C). Then apply the colouring paper with the printed side to the hot plastic.

Tip: Leave a small corner of the paper sticking up so you can easily grab it to peel it off.

After only 15 seconds, you can peel off the paper again and continue processing the plate.

Please note

If necessary, the plastic plate can be reheated in the oven after colouring. This will cause the design to lose some of its sharpness and colour intensity.

To transfer the design onto white lamination fabric, polyester strap, hook or loop strap, you will need a standard iron and a suitable surface.

Set the iron to the highest temperature and iron on the design.

Allow for a contact time of 15 seconds.

Colouring paper configurator



It's so easy to find your favourite design!
Scan the QR code here and try it out right away.

First select the orthosis you need and then click through our wide range of patterns!

www.streifeneder.com/op/configurator

Streifeneder ortho.production GmbH

Moosfeldstraße 10 · 82275 Emmering · Germany
T +49 8141 6106-0 · F +49 8141 6106-50
office@streifeneder.de
www.streifeneder.com/op

This catalogue is protected by copyright.

Reproduction of the complete catalogue, even in abstracts, in any form or by any means is only permitted with explicit written authorisation of the Streifeneder ortho.production GmbH.

The indicated measurements and weights are guide values without obligation. The manufacturer reserves the right to make changes to the design or shape, deviations in colour and changes to the scope of delivery during the delivery period, provided that the object of purchase is not significantly changed and the changes are reasonable for the purchaser.

After the editorial deadline of the catalogue, 01.07.2022, changes to the product may occur.

The general terms and conditions apply, they can be viewed any time at www.streifeneder.com/op.

100W202/E 10/2022



www.streifeneder.com/op

